


BLOEDBESTRALING INDICATOREN

ad-sure™ chromicVue™
Good irradiation indicator

chromicVue™ 15 Gy (x25)

DC15G01

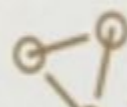

Ashland

Table of contents

Ashland **3**

Rad-Sure Blood Irradiation Indicators - Ashland 4

..... 6



Ashland is een wereldwijde leider in speciale materialen en biedt innovatieve oplossingen die de veiligheid, precisie en resultaten voor patiënten in verschillende medische disciplines verbeteren. Hun portfolio ondersteunt zorgverleners op het gebied van bestralingstherapie, diagnostische beeldvorming, wondverzorging en regeneratieve geneeskunde.

Product offering

Rad-Sure Blood Irradiation Indicators - Ashland



Rad-Sure Blood Irradiation Indicators - Ashland

Rad-Sure Blood Irradiation Indicators give visual verification of irradiation. When they are attached to blood products, they show whether the blood products have been irradiated or not. Rad-Sure is the standard for blood irradiation indicators, and it has been for over 25 years.

Hospitals irradiate blood to prevent a rare fatal complication of blood transfusion (TA-GVHD).

PEO provides two kinds of blood irradiation indicators. The first one is for gamma ray and the second one for x-ray. There are also two options for the minimum dose: 15 Gy or 25 Gy.



HOW DOES IT WORK?


LOOK FOR THE “NOT”


peel – stick – irradiate – read

When a doctor attaches an irradiator to a blood product, the indicator shows whether the blood is irradiated or not. Before the indicator is attached, it reads: “not irradiated”. Once you attach the indicator to irradiated blood, the word “not” disappears and the indicator reads: “irradiated” (as you can see here).

The Rad-Sure indicators adhere to AABB standards and hold the [AABB seal of compliance](#).



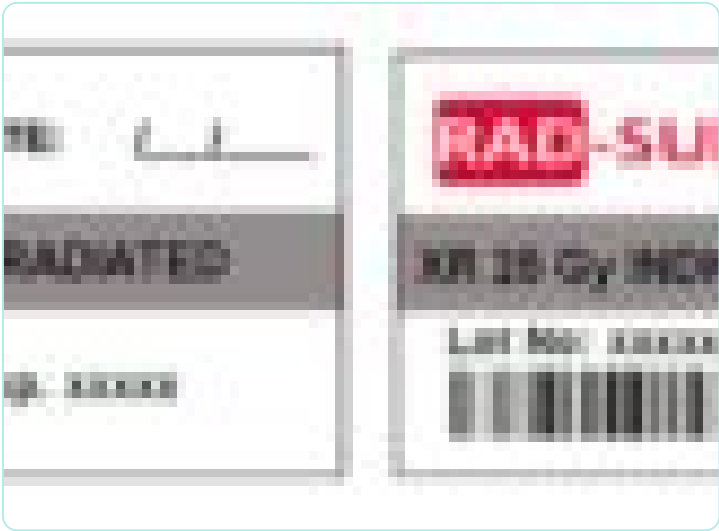

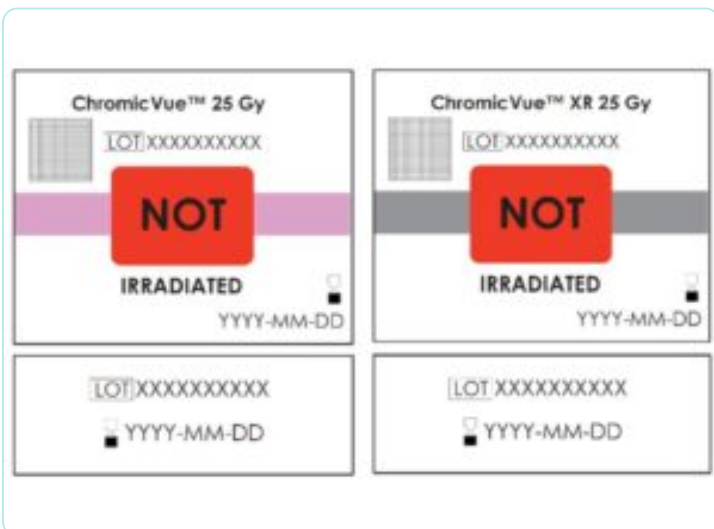

dependable features:

- film-based indicator – made with highly accurate Gafchromic™ film used in radiation oncology centers around the world
- less subjective – the product has been properly irradiated when the “NOT” is completely obscured
- indicators can be stored at room temperature
- Color-blind friendly – no need to match colors
- 30 years of reliable film technology

Feature	Benefit
compact format	• optimal for any blood unit including neonates • efficient use allows for more space on table
2D barcode	• 2D barcode reads fast & with expiration date • legacy 1D barcode still available for use
safe box	• indicators dispensed only at a time • no refrigeration – indicators remain potent • sustainable packaging – boxes are recyclable • smaller box format allows for reduced storage
expiration date	• additional 30 indication per box
expiry date small &	• for optional use in logbooks and general d